


SlapDriver Interference Screwdriver

Product Highlight



The **SlapDriver interference screwdriver** family was designed to combine the technology and reliability of our hexalobe, trilobe, and quick connect ratcheting screwdrivers with the convenience of a built-in slap-hammer mechanism to make driver removal easier and faster.

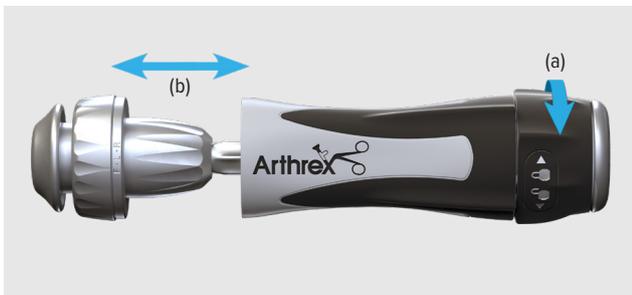

(a) The SlapDriver screwdriver features a convenient twist-lock knob on the back of the handle to access the slap hammer feature. **(b)** When the lock feature is disengaged, the slap hammer mechanism will be free to piston, helping remove the driver from an implanted screw.

Features and Benefits

Built-in Slap Hammer


- Saves time as a mallet is no longer needed to disengage the driver from the interference screw
- Reduces wear and tear on driver handles

Improved Ergonomics and Performance

- Handles use ergonomic, four-sided design to improve screw insertion
- Smooth quick connect mechanism allows for easy connect and disconnect of driver shaft to handle

Versatility


- Available in four versions to accommodate surgeon's preferences
- Configured as ratcheting with quick connect or nonratcheting fixed versions

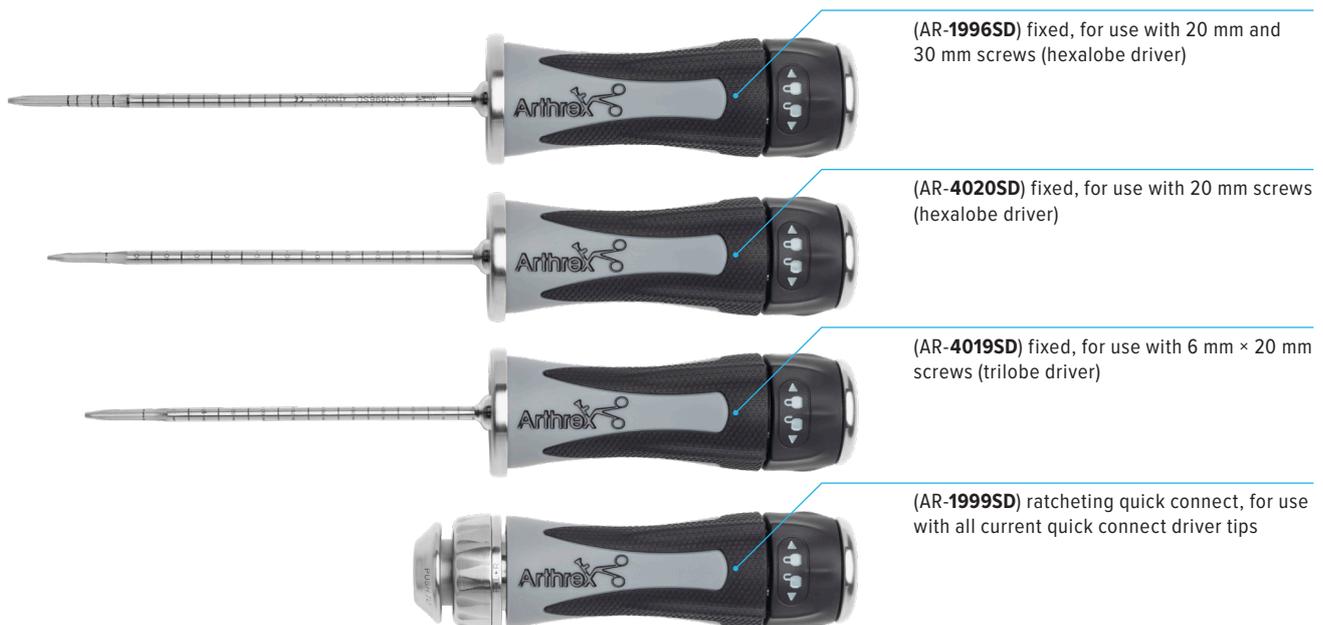


Arthrex®

SlapDriver Interference Screwdriver

Ordering Information

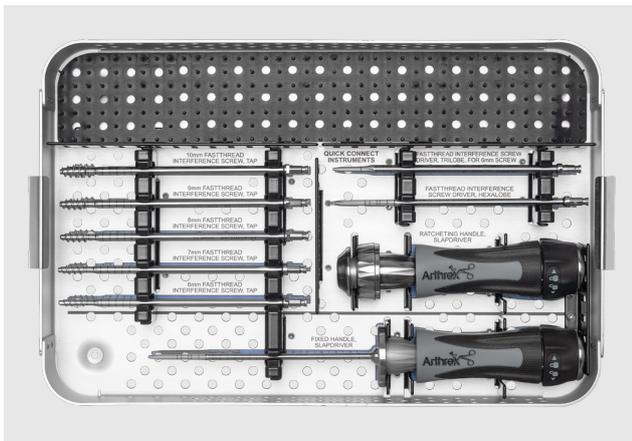

FastThread Interference Screw Instrument Set (AR-1996S)

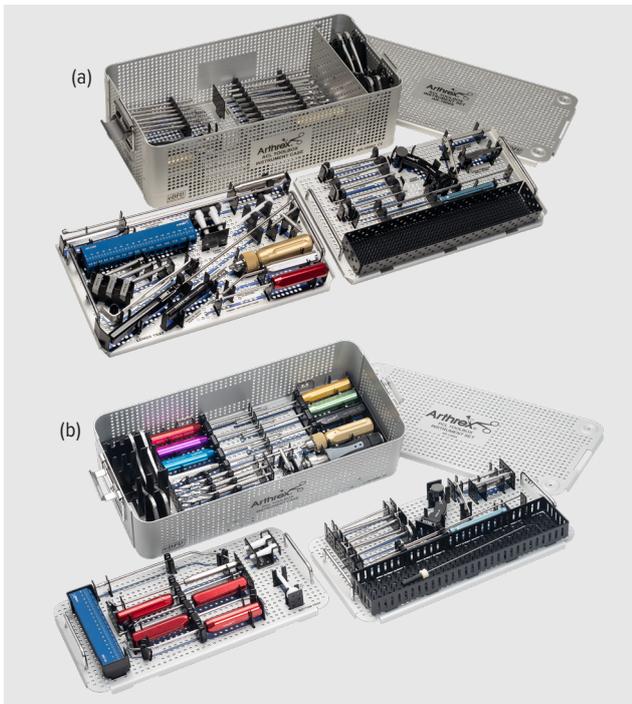

ACL and PCL ToolBox Set (a) AR-1900S, (b) AR-1269S

SlapDrivers screwdrivers are included in the recently revised ACL and PCL ToolBox instrumentation sets (AR-1900S and AR-1269S) as well as the FastThread™ interference screw instrumentation set (AR-1996S).

Instruments

Product Description	Item Number
Ratcheting SlapDriver	
SlapDriver, ratcheting quick connect handle	AR-1999SD
Quick Connect Drivers	
Quick Connect Driver, for 20 mm and 30 mm screws (hexalobe)	AR-1996CD-1
Quick Connect Driver, for 20 mm-length screws only (hexalobe)	AR-4020D-1
Quick Connect Driver, extended-length shaft (hexalobe)	AR-1996CDL-1
Flexible Quick Connect Driver, for 20 mm-length screws only (hexalobe)	AR-4020DF
Quick Connect Driver, for 6 mm-diameter screws only (trilobe)	AR-4019D-1
Fixed SlapDrivers	
SlapDriver, fixed, for 20 mm and 30 mm lengths only (hexalobe)	AR-1996SD
SlapDriver, fixed, for 20 mm-length screws only (hexalobe)	AR-4020SD
SlapDriver, fixed, for 6 mm-diameter screws only (trilobe)	AR-4019SD