

Description

Energel wound hydrogel is a sterile, water-soluble gel formulated to maintain a moist wound environment and provide moisture to a dry wound or surgical incision.

Indications

Energel hydrogel is intended to cover a wound to protect against abrasion, friction, desiccation, and contamination.

Product Features and Specifications

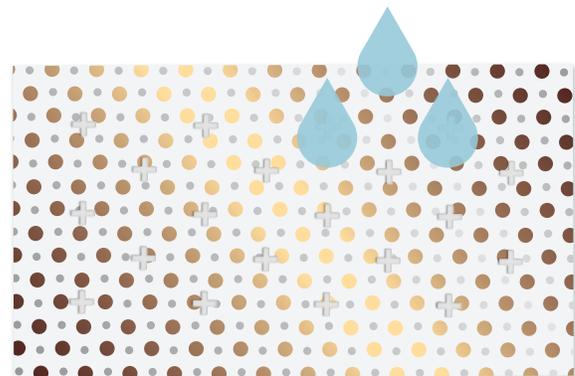
- Double-packaged sterile for use in the operating room
- Optimally sized for single use (7.5 g / 0.26 oz)
- 10 sachets per box
- Compatible with JumpStart dressings, which are powered by patented V.Dox™ technology
- Use Energel hydrogel to activate JumpStart dressing's embedded microcell batteries
- Maintains JumpStart dressing's conductivity for up to 7 days

Application

- Cleanse wound and surrounding skin prior to application.
- Hydrate dressing with a thin, even layer of Energel hydrogel and place onto wound, or apply enough Energel hydrogel to cover the wound base.
- Apply appropriate cover dressing(s) to help manage wound drainage and maintain a moist wound environment.

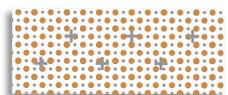
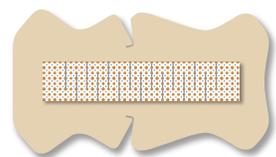
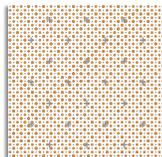
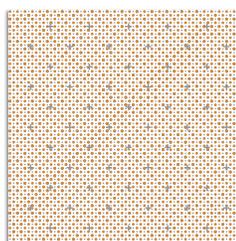
To Use With JumpStart Dressings

- Apply a thin, even layer of Energel hydrogel on the dotted side of the wound dressing.
- Place the dressing, dots down, directly onto the wound.
- Gently press down to ensure direct wound contact.



Energel[®] Wound Hydrogel (Compatible With JumpStart[®] Antimicrobial Wound Dressings)

Suggested Application Volumes of Energel Hydrogel

	Single Layer Dressings	Composite Dressings
<1/4 sachet	 <p>1 in × 1 in 2.5 cm × 2.5 cm 2 in × 2 in 5 cm × 5 cm</p>	 <p>2.5 in ø 6 cm ø 4 in ø 10 cm ø 4 in × 4 in 10 cm × 10 cm</p>
1/4 sachet	 <p>1.5 in × 8 in 3.5 cm × 20 cm 2 in × 5 in 5 cm × 12.5 cm 3 in × 3 in 7.5 cm × 7.5 cm 1.5 in × 10 in 3.5 cm × 25 cm 4 in × 4 in 10 cm × 10 cm</p>	 <p>5 in × 6 in 12.5 cm × 15 cm 4.5 in × 10 in 11 cm × 25 cm 4.2 in × 7.5 in 10.7 cm × 19 cm 4.4 in × 9.6 in 11.2 cm × 24.4 cm</p>
3/4 sachet	 <p>4 in × 8 in 10 cm × 20 cm</p>	<p>1/2 sachet</p>  <p>6 in × 11.5 in 15 cm × 29 cm</p>
1 sachet	 <p>8 in × 8 in 20 cm × 20 cm</p>	
2 sachets	 <p>12 in × 12 in 30.5 cm × 30.5 cm</p>	

Note: These are approximate amounts of Energel hydrogel to use when dressing a wound, based on size of the selected dressing. You may choose to add more or remove a small amount, as needed, depending upon the condition of the wound and to ensure a thin, even layer over the dotted surface of the dressing.