

# Synergy<sup>RF</sup>™ AR-9800


Quick Start Guide



## Unpacking

Carefully unpack and inspect all components for shipping damage. Save the packaging for later transport of the device. The footswitch is provided separately. The probe is provided separately and is in sterile packaging.

Users must read the Synergy<sup>RF</sup> User's Manual and Apollo<sup>RF</sup> Probe DFU prior to using these devices. The User Manual and DFU provide the indications and contraindications for use, warnings, symbol definitions, and the full list of product instructions. These documents can be found on [Arthrex.com](http://Arthrex.com).

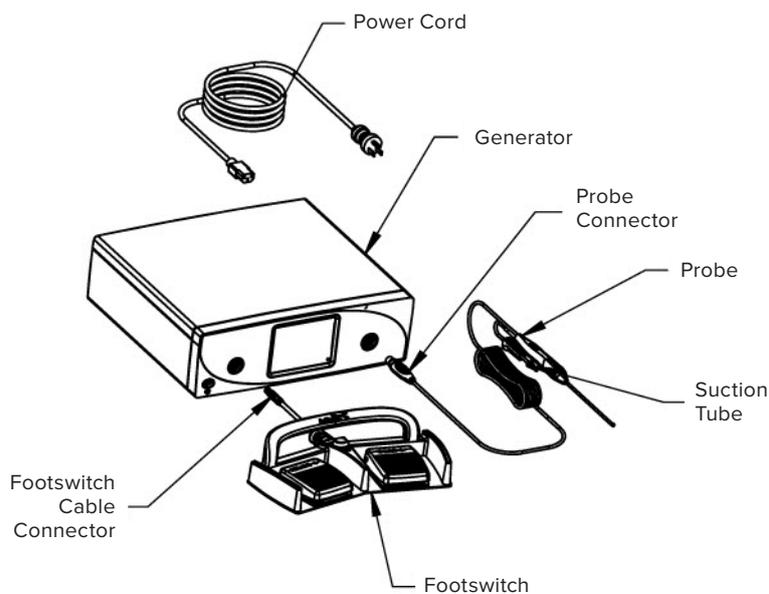

## Setup



1

Place console on a flat, dry surface



2

Connect power cord to console




3

Connect power cord to outlet



4

Power on console

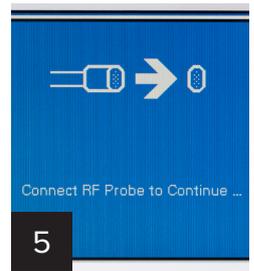

5

Allow console to fully initialize




6

Attach probe via connector to console



7

For suction probes, attach suction tube

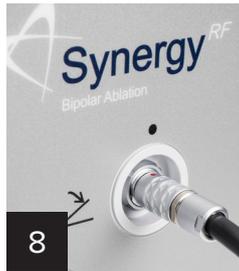

8

Attach footswitch if desired



9


Change settings using touch panel to set main screen display

## To Adjust Settings



1

Press menu setting button

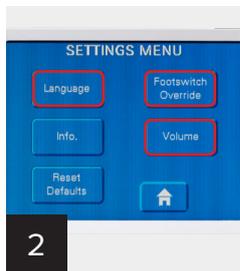

2

Press desired setting



**For Language:**  
Select desired language and then do one of the following:

Press **HOME** to save setting and return to main screen  
-OR-  
Press **Setting Button** to save setting and return to **MENU**  
-OR-  
Press **X** to return to **MENU** without changing default



**For Footswitch Override:**  
Select desired option and then do one of the following:

Press **HOME** to save setting and return to main screen  
-OR-  
Press **Setting Button** to save setting and return to **MENU**  
-OR-  
Press **X** to return to **MENU** without changing default



**For Volume:**  
Select desired volume using + or – buttons and then do one of the following:

Press **HOME** to save setting and return to main screen  
-OR-  
Press **Setting Button** to save setting and return to **MENU**  
-OR-  
Press **X** to return to **MENU** without changing default

## Button Activation

1. Ablation power setting
2. Ablate
3. Coag



## Troubleshooting: Faults and Solutions

Fault	Solution
Unit does not power on	<ol style="list-style-type: none"> <li>1. Check the main power plug and wall receptacle.</li> <li>2. Check the main fuses per the User Guide instructions.</li> <li>3. Call Arthrex Tech Support at <b>1-888-420-9393</b>.</li> </ol>
Accessory Apollo <sup>RF</sup> probe does not work	<ol style="list-style-type: none"> <li>1. Disconnect and reconnect probe.</li> <li>2. Turn off console, wait 30 seconds, repower console.</li> <li>3. If applicable, disconnect and reconnect footswitch, replace if footswitch not functioning.</li> <li>4. Open new Apollo<sup>RF</sup> probe.</li> </ol>
Yellow fault message	Refer to User Guide for proper disposition.
Red fault message (Critical Failure)	<ol style="list-style-type: none"> <li>1. Make note of message displayed.</li> <li>2. Call Arthrex Tech Support at <b>1-888-420-9393</b>.</li> </ol>

**To return or repair device: Contact Arthrex Product Surveillance at 1-866-267-9138.**

**Arthrex, Inc.**  
 1370 Creekside Blvd.  
 Naples, FL 34108-1945 USA  
 Toll Free: 1 (800) 934-4404  
 www.arthrex.com

**Arthrex GmbH**  
 Erwin-Hielscher-Strasse 9  
 81249 München, Germany  
 Tel: +49 89 909005-0  
 www.arthrex.de

Arthrex Tech Support:  
 +1-888-420-9393

+49 89 90 90 05 8800