


C-Mount Wrench User Guide

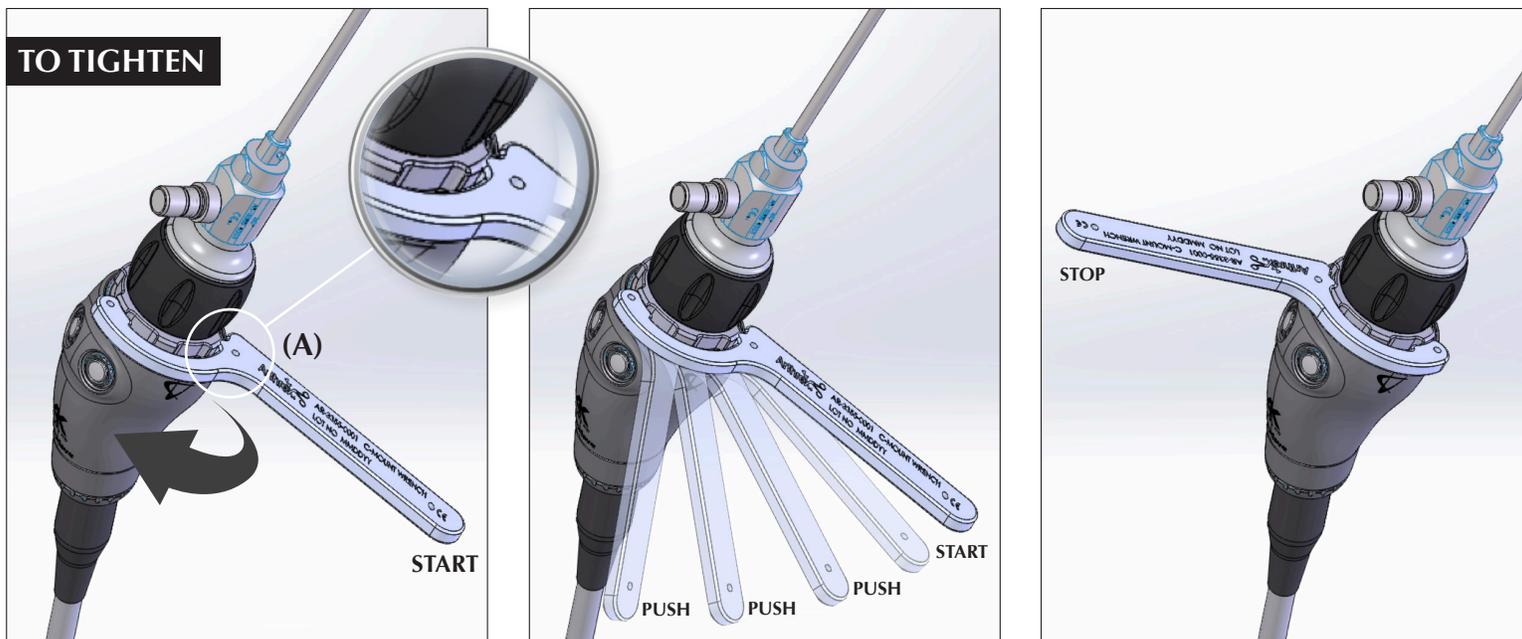

To tighten the scope on the head: Screw the scope on to the Synergy^{UHD4™} C-mount camera head by hand until tight. Place the wrench on the silver nut (A) on the scope. While holding the camera head firmly, **PUSH the wrench clockwise** to tighten the scope on the camera head.



To remove the scope from the head: Place the wrench on the silver nut (A) and push the wrench counterclockwise until the scope is loosened. Continue unscrewing by hand to remove the scope.

AR-3355-0001 C-Mount Wrench