


Hip Labral Scorpion™


Simplified Single Portal Suture Passing and Retrieving




Hip Labral Scorpion
AR-16991

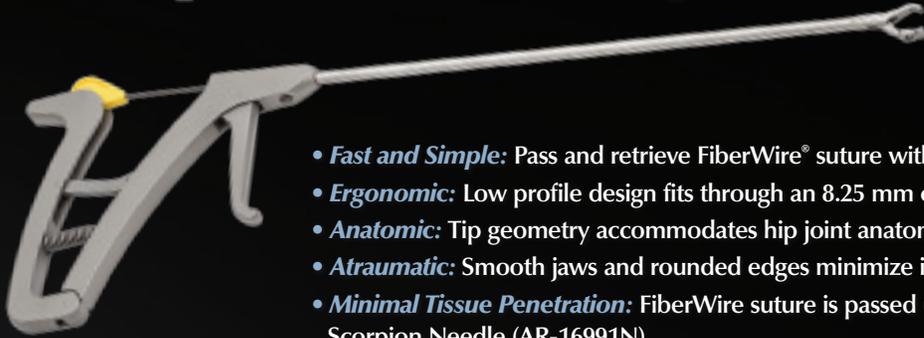

- **Fast and Simple:** Pass and retrieve FiberWire® suture with minimal steps
- **Ergonomic:** Low profile design fits through an 8.25 mm cannula
- **Anatomic:** Tip geometry accommodates hip joint anatomy
- **Atraumatic:** Smooth jaws and rounded edges minimize iatrogenic damage
- **Minimal Tissue Penetration:** FiberWire suture is passed using a small diameter Hip Labral Scorpion Needle (AR-16991N)

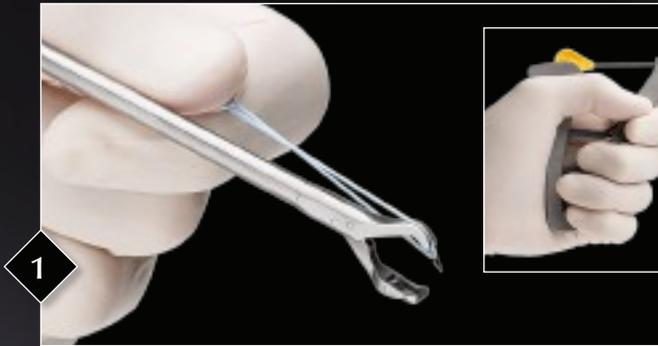

1
Load 1-2 inches of FiberWire into the groove on the top jaw and squeeze the instrument until the FiberWire enters the slot on the needle.

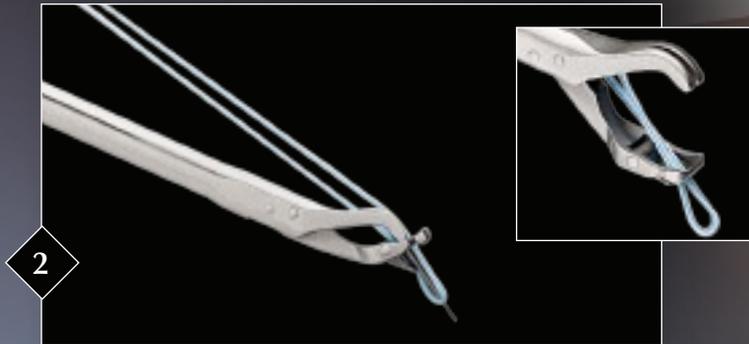

2
Grab the labrum and squeeze the handle to advance the needle and FiberWire through the tissue.



3
Retract the Scorpion from the tissue with the captured FiberWire in the lower jaw. To release the suture, pull both suture limbs and squeeze the instrument.



Arthrex®

www.arthrex.com

... up-to-date technology
just a click away

U.S. PATENT NO. 6,716,234 and PATENT PENDING

© 2014, Arthrex Inc. All rights reserved.

LS1-0327-EN_A